
i-OIL

EN	Dental handpieces heads oil
LT	Universalus tepalas visų rūšių odontologiniams antgaliams
DE	Zahnärztliche Handstücke Köpfe Öl
BG	Масло за глави на стоматологични накрайници
CS	Zubní násadce, hlavy, olej
DA	Dental håndstykker hovedolie
ET	Õli hambaravi käsiinstrumentidele
FR	Huile pour têtes de pièces à main dentaires
EL	Λάδι λιπαρών οδοντικών εργαλείων
HU	Fogászati kézszerszámfej olaj
IT	Testine per manipoli dentali olio
LV	Zobārstniecības rokas instrumentu galvīņu eļļa
NO	Salve for håndstykker tanntips
PL	olej do głowic dentystycznych
PT	Óleo de lubrificação de acessórios dentário
RO	Ulei pentru capetele turbinelor
SK	Zubné násadce, hlavy, olej
SL	Mazivno olje za zobozdravstvene pripomočke
ES	Aceite lubricante para piezas de mano dentales
SV	Tandläkarhandstycken, huvuden, olja
TR	Dental piyasemen başlıkları için yağ
RU	Масло для стоматологических наконечников
UA	Масло для стоматологічних наконечників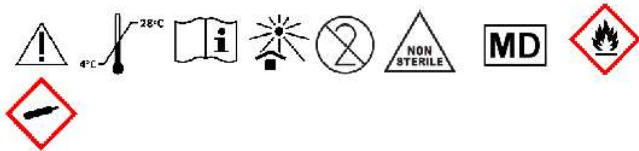

INSTRUCTION FOR USE EN

DESCRIPTION

i-OIL is non-sterile universal oil which lubricates inner parts of instruments and prolongs the working life of dental handpieces.

COMPOSITION

Gases (liquefied) 50-70%, white mineral oil 30-50%, excipients. Product does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

spray pattern	jet
reliability	non-blocking

INTENDED PURPOSE AND CLINICAL BENEFITS

Intended for use during maintenance dental handpieces for lubrication of all kind of dental handpieces.

CLINICAL INDICATIONS

- For the internal mechanisms of dental handpieces and air motors and general handpiece maintenance.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

None known.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, product may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract irritation). Do not use product if patient is allergic to product or to any of the ingredients.

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. They may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Indirect contact with part of the body – mouth.

INTENDED USER

i-OIL developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental materials. There is no need for specific training.

STERILITY

i-OIL is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

i-OIL is designed to be used in dental office where ambient temperature is 18-25°C. Do not re-use.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumable component such as special nozzle for handpieces is supplied with the device.

INSTRUCTION FOR USE

1. Hold bottle in a vertical position.
2. Spray for one second into the bottom of handpiece after each use and before sterilization.
3. Clean handpiece with paper towel.

WARNINGS

i-OIL contains extremely flammable gases. Pressurized container: Do not pierce or burn, even after use. Keep away from sources of ignition. Keep away from heat/sparks/open flames/hot surfaces. Protect from sunlight and do not expose to temperatures exceeding 50C/122F. May explode while heated.

PRECAUTIONS

Do not breathe vapour-spray. Use only in a well-ventilated area. Wear protective gloves/protective clothing/eye protection/face protection. IF exposed or concerned: Call a POISON CENTER/doctor.

SHELF-LIFE

Shelf-life of i-OIL is 5 years from the date of manufacture. Do not use after the expiry date. See packing for expiry date. Do not re-use.

STORAGE

Keep product tightly closed in dry well-ventilated place at 4-28°C. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Protect from direct sunlight. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

i-OIL is safe and performs as intended if it is used in accordance to manufacturer's instruction for use.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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