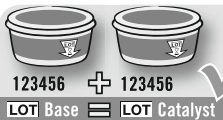
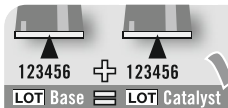
Ⓓ	Gebrauchsanweisung	5
Ⓖ	Instructions for use.....	13
Ⓕ	Mode d'emploi	21
Ⓔ	Instrucciones de uso	29
Ⓘ	Istruzioni per l'uso	37
Ⓟ	Instruções de uso	45
Ⓝ	Gebruiksaanwijzing	53
Ⓔ	Bruksanvisning	61
Ⓓ	Brugervejledning	69
Ⓝ	Bruksanvisning	77
Ⓕ	Käyttöohjeet.....	85

A

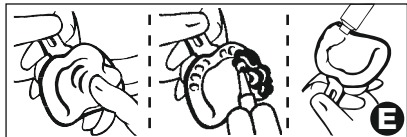
B

C

D

E

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Stand: 2014-08

Description of applicational/packaging icons

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D = Mischzeit

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K = Intraoral Tip

Elastomeric, silicone-based precision impression material, addition-cured.

The Flexitime Fast & Scan range is an optimally matched system and its different types of material can be combined for all impression techniques to suit specific requirements.

Material

Flexitime Easy Putty

Flexitime Dynamix Putty

Flexitime Heavy Tray

Flexitime Dynamix Heavy Tray

Flexitime Medium Flow

Flexitime Light Flow

Pack

Can, 1:1, 2 x 300 ml

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

Cartridge, 1:1, 50 ml

The Flexitime Fast & Scan range products are prepared without requiring additional surface treatment for optical scanning in dental scanners designed for scanning impression materials, such as the 3 Shape D 700 laser scanner.

Side effects

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

Notes on use

Not to be used in conjunction with other silicone materials.

Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip.

After use, leave the mixing tip on the cartridge to keep it closed. Immediately before the

next use, remove the mixing tip and attach a new one. Mixing Tips and Intraoral Tips are

single-use products. To prevent cross-contamination between patients, Intraoral Tips must

be discarded after each use.

In all addition-curing silicone impression materials, the setting process may be affected by contact with products containing active sulphur or Nitrogen, or by fresh composite fillings or superstructures. These products include latex gloves, polysulphide impression materials, retraction solutions, etc.

Flexitime impression materials should be stored at room temperature 23°C (73°F) for processing. Polyvinyl siloxanes are chemically resistant. Material which has not set can soil clothing. Visually inspect for damage before use. Damaged products must not be used.

Flexitime [®] <i>Fast & Scan</i>	Consistency	Product colour		Mixing time	Extraoral Working time* 23°C (73°F)	Intraoral Working time 35°C (95°F)	Min. time in mouth	Accessories	
		Base	Catalyst					Mixing Tips	Intraoral Tips
	EN ISO 4823			s	min:s	min:s	min:s		
Flexitime [®] <i>Fast & Scan</i> Easy Putty	Typ 0 mouldable	green	white	30	1:30	—	2:00	—	—
Flexitime [®] <i>Fast & Scan</i> Dynamix Putty	Type 0 mouldable	green	white	auto	1:30**	—	2:00	dynamic yellow 66035999	Na
Flexitime [®] <i>Fast & Scan</i> Heavy Tray	Typ 1 heavy bodied	green	white	auto	1:30	—	2:00	green 66001989	—
Flexitime [®] <i>Fast & Scan</i> Dynamix Heavy Tray	Typ 1 heavy bodied	green	white	auto	1:30**	—	2:00	dynamic yellow 66035999	Na
Flexitime [®] <i>Fast & Scan</i> Medium Flow	Type 2 medium bodied	yellow	white	auto	1:30	0:30	2:00	yellow 66001988	yellow 66000782
Flexitime [®] <i>Fast & Scan</i> Light Flow	Type 3 light bodied	orange	white	auto	1:30	0:30	2:00	yellow 66001988	yellow 66000782

* at room temperature of 23°C (73°F), 50 % rel. humidity. Higher temperatures shorten, lower temperatures prolong these times.

** measured using the Dynamix[®] mixer from Kulzer

The data above is accurate to the best of our knowledge and is based on internal test results.

Optical scanning:

The quality of the result depends on the impression scanner. Follow the manufacturer's recommendations for preparation of teeth and the instructions for the optical scanning procedure.

Model fabrication:

The impression can be cast 30 minutes after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.

Recommended model material:

All standard model materials can be used.

We recommend:

Class III plaster, MOLDANO®

Class IV plaster, MOLDASTONE®/MOLDASYNT®

Electroplating:

Impressions can be electroplated with copper and silver in conventional baths.

Disinfection:

Impressions can be treated with an aqueous sodium hypochlorite solution (5.25 %, for up to 10 minutes*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.

*(DGZMK 3/93 V1.0)

Adhesion:

We recommend using our Universal Silicone Adhesive.

Storage conditions:

Do not use Flexitime Fast & Scan impression materials if the expiry date has elapsed.
Do not store above 25°C (77°F).

Dynamix system, 380 ml – step by step

Fig. 1
Remove cap
Insert cartridge
into the mixer

Fig. 2
Attach Dynamix
Mixing Tip

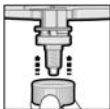
Fig. 3
Position Dynamix
fixation ring

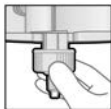
Fig. 4.1
Attach Dynamix
fixation ring
Slide over
recesses

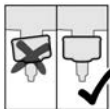
Fig. 4.2

Fig. 5
Lock by giving
a ¼ turn
clockwise

Inserting the Dynamix cartridge:

For information on how to handle the mixer being used please refer to the detailed instructions issued by the relevant manufacturer.

1. Hold the tab on the cap of the outlets and bend it upwards through 90° without applying any excessive force.
2. Take the tab between your thumb and index finger whilst you hold it securely with your other hand and remove the cap from the cartridge completely. (Fig. 1)
3. Place one of the enclosed Dynamix Mixing Tips over the outlets of the cartridge. (Fig. 2)
4. If it proves difficult to attach the Dynamix mixing tip, check to make sure that the central internal hexagon of the Dynamix mixing tip is properly aligned with the hexagon of the drive spindle.
5. When the Dynamix mixing tip is in the correct position (Fig. 3), push the Dynamix fixation ring over the Dynamix mixing tip until it stops and turn it clockwise (1/4 turn) until it is securely locked in place. (Figs. 4.1, 4.2 and 5)
6. When the material has been dispensed, retract the mixer plungers from the cartridge in order to avoid continuous pressure and thus prevent excess material from continuing to emerge from the Dynamix mixing tip.

Notes about safe use of the Dynamix System

Dynamix cartridges are heavy and can be damaged if they are dropped.

Damaged cartridges must no longer be used.

Only use Dynamix cartridges in mixers with a closed protective flap.

Before using for the first time, adjust the levels of the two material components. Dispense approx. 2 – 3 cm material and discard this amount before starting to fill the tray. When the cartridge is used in different mixers, 2 – 3 cm of material has to be discarded after each device change. Each time before you use the material check the outlet channels of the cartridge for blockages.

At the end of a working day or if the mixer is not going to be used for a lengthy period, remove the cartridge from the device and store it in a horizontal position with the outlets closed off by the mixing tip or cap.

After use, leave the Dynamix mixing tip on the cartridge to keep it closed. Immediately before the next use, remove the Dynamix mixing tip and attach a new one. To do so, detach the Dynamix fixation ring and use it again for the new mixer attached.

Only use Dynamix System components for dispensing and mixing the impression material in Dynamix cartridges.

Check the system components for physical damage on a regular basis and only use components which are intact.

Empty cartridges must be disposed of safely in accordance with the EU directive 94/62/EC.

Warnings and safety instructions

If material comes into contact with the eyes, rinse it out with water for several minutes. If symptoms persist, consult an eye specialist.

Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases.

Only to be used by dentists and for its intended use.

Dated: 2014-08

Description of applicational / packaging icons

A = Only use pastes with same LOT No

B = Icon "Use of the special program at the
Dynamix® *speed* Mixing Unit possible"

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Kulzer GmbH
Kulzer, LLC