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ENGLISH

General

Filetek™ Supreme XT Flowable Restorative, manufactured by 3M ESPE, is a low viscosity, visible-light activated, radiopaque, flowable nanocomposite based on 3M™ ESPE™ Filetek™ Supreme XT Universal Restorative. The restorative is packaged in syringes and available in a variety of tooth-colored shades. The shades offered with Filetek Supreme XT flowable restorative were designed to coordinate with shades offered with Filetek Supreme XT Universal Restorative. Filetek Supreme XT flowable restorative contains bisGMA, TEGDMA and bis-EMA resins. The filler in Filetek Supreme XT flowable restorative is a combination of non-agglomerated/non-aggregated, 75nm silica nanofiller, non-agglomerated/non-aggregated, 15-20nm zirconia nanofiller and loosely bound agglomerated zirconia/silica nanocluster, consisting of agglomerates of 5-20nm primary zirconia/silica particles. The cluster particle size range is 0.6 to 1.4 microns. The inorganic filler loading is approximately 65% by weight (55% by volume).

Indications

- Restoration of minimally invasive cavity preparations (including Class III and V restorations and occlusal pits and fissures)
- Base/liner under direct restorations

- Repair of small defects in esthetic indirect restorations
- Pit and fissure sealant
- Undercut blockout

- Repair of resin and acrylic temporary materials

Precautions for Patients and Dental Personnel Precautionary Information for Patients:

Avoid use of this product in patients with known acrylic allergies. This product contains substances that may cause an allergic reaction by skin contact in certain individuals. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

Precautionary Information for Dental Personnel:

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed. Protective eyewear for patients and staff is recommended when using dispensing tip.

Instructions for Use

I) Preliminary

A. Prophyl: Teeth should be cleaned with pumice and water to remove surface stains.

B. Shade Selection: Prior to isolation of tooth, select the appropriate shade(s) of Filetek Supreme XT flowable restorative using a standard VitaPan Classic shade guide. Shade selection accuracy may be enhanced via a mock-up; place the chosen shade of the restorative material on the unetched tooth. Manipulate the material to approximate the thickness and site of the restoration. Cure. Evaluate the shade match under different lighting sources. Remove the restorative material from the unetched tooth with an explorer. Repeat process until an acceptable shade match is achieved.

For sealants, a contrasting shade may be desirable to enhance detection.

C. Isolation: A rubber dam is the preferred method of isolation. Cotton rolls and an evacuator can also be used.

II) Directions

A. Isolate and dry the preparation.

B. Pulp protection: If a pulp exposure has occurred and the situation warrants a direct pulp capping procedure, use a minimum amount of calcium hydroxide on the exposure followed by an application of Vitrebond™ Light Cure Glass Ionomer, manufactured by 3M ESPE. Vitrebond may also be used in tire areas of deep cavity excavation.

C. Adhesive System: Follow manufacturer’s instructions regarding placement.

Note: Silane treatment is recommended for repair of ceramic restorations followed by the adhesive application.

D. Dispensing: Filetek Supreme XT flowable restorative can be delivered directly from the dispensing tip.

1. Protective eyewear for patients and staff is recommended when using the dispensing tip.

2. Prepare the dispensing tip: Remove cap and SAVE. Twist the prebent disposable dispensing tip securely onto the syringe. Holding the tip away from the patient and any dental staff, express a small amount of Filetek Supreme XT flowable restorative to assure that the delivery system is not plugged.

3. If plugged, remove the dispensing tip and express a small amount of Filetek Supreme XT flowable restorative directly from the syringe. Remove any visible plug, if present, from the syringe opening. Replace dispensing tip and again express composite. Filetek Supreme XT flowable restorative may be extruded onto a dispensing pad and applied with a brush or other appropriate instrument.

E. Placement:

1. Place and light cure restorative in increments as indicated in Section F.

2. Avoid intense light in the working field.

3. To prevent oozing of paste from the syringe tip after placement, pull back on syringe plunger.

F. Curing: Filetek Supreme XT flowable restorative will cure only by exposure to light. Cure each increment by exposing its entire surface to a high intensity visible light source, such as a 3M ESPE

curing light. Hold the light guide tip as close to the restorative as possible during light exposure.

Shade	Increment depth	Cure time
Opaque A3	2.0mm	40 sec
All other shades	2.0mm	20 sec

G. Complete the restoration:

1. Base/Liner application: place a composite restorative material, such as Filetek™ Supreme XT Universal Restorative, manufactured by 3M ESPE, directly over the cured Filetek Supreme XT flowable restorative. Follow the manufacturer’s instructions regarding placement, curing, finishing, occlusal adjustment, and polishing.

2. Direct Restorative Application:

a) Contour restoration surfaces with fine finishing diamonds, burs or stones.

b) Check occlusion with a thin articulating paper. Examine centric and lateral excursion contacts. Carefully adjust occlusion by removing material with a fine polishing diamond or stone.

c) Polish with Sof-Lex™ Finishing and Polishing System, manufactured by 3M ESPE, or with white stones, rubber points or polishing paste where discs are not suitable.

3. Pit and Fissure Sealants: gently remove the inhibited layer remaining after light-curing with a slurry of pumice or polishing paste.

III) Storage and Use:

A. Filetek Supreme XT flowable restorative is designed for use at room temperature of approximately 21-24°C or 70-75°F. Shelf life at room temperature is 2 years. See outer package for expiry date.

B. Do not expose restorative materials to elevated temperatures or intense light.

C. Storage in refrigerator ensures longest possible shelf life. Allow to reach room temperature prior to use.

D. Do not store materials in proximity to eugenol containing products.

E. Disinfection: Discard used syringe tip and replace with syringe storage cap. Disinfect the capped syringe using an intermediate level disinfection process (liquid control) as recommended by the CDC and endorsed by the ADA. Guidelines for Infection Control in Dental Health-Care Settings – 2003 (Vol.52; No. RR-17), Centers for Disease Control and Prevention.

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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VITAPAN® is a trademark of VITA Zahnfabrik.
3M, ESPE, Filtek, Vitrebond and Sof-Lex are trademarks of 3M or 3M ESPE AG. © 3M 2005. All rights reserved 44-0007-4337-5-A