# i-HYDRO®

<b>EN</b>	Hydrofluoric acid porcelain etching gel
<b>ES</b>	Gel de grabado de porcelana de ácido fluorhídrico
<b>LT</b>	Hidrofluoro rūgštis porceliano ėsdinimo gelis
<b>DA</b>	Flussyregel til ętsning af porcelęn
<b>NO</b>	Flussyre porselen etsegel
<b>SV</b>	Etsningsgel av porslin med fluorvętesyra

## INSTRUCTION FOR USE EN

### DESCRIPTION

i-HYDRO is a 9% hydrofluoric acid gel which provides surface roughness and ensures strong mechanical bond between porcelain and the resin matrix of the composite.

### INDICATIONS

- For etching of porcelain restorations (crowns, bridges, inlays, onlays) prior cementation.

### CONTRA-INDICATIONS

Application in the oral cavity is contraindicated.

### INTENDED USER

i-HYDRO is developed for professional use in dentistry only. Its user only licensed doctor/technician who has knowledge how to use common dental materials. There is no need for specific training.

### STERILITY

i-HYDRO is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

### USE ENVIRONMENT

i-HYDRO is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of material is suitable for single use (only for one preparation). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

### CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the product. Consumables, such as application tips, are supplied with the product.

### INSTRUCTION FOR USE

- Remove i-HYDRO syringe cap.
- Carefully attach application tip to the syringe.
- Delicate push on plunger and apply layer of gel on all surfaces needed to be etched. Do not force plunger.
- Leave gel in place for 90 seconds.
- Rinse gel with copious amount of water and dry porcelain surface with air. Etched surface should become frosted.

### WARNINGS

Never force syringe plunger. Do not apply i-HYDRO onto glazed porcelain surfaces because they will lose their gloss. Over etching may weaken porcelain strength. Close syringe with cap immediately after use. Dispose of used or contaminate application tips.

### PRECAUTIONS!

i-HYDRO contains 9% hydrofluoric acid. Hydrofluoric acid causes severe skin burns and eye damage, fatal if swallowed, inhaled and in contact with skin.

IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. Do not get in eyes, on skin, or on clothing. IF ON SKIN OR MUCOSA: Wash with plenty of water. Take off all contaminated clothing and wash it before reuse. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IN ALL CASES: Immediately call a POISON CENTER/doctor.

It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and the patient. Do not breathe vapours. In case of inadequate ventilation wear respiratory protection. Wash hands thoroughly after handling.

### SHELF-LIFE

Shelf-life of i-HYDRO is 2 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

### STORAGE

Keep container tightly closed in dry well-ventilated place at 10-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of reach of children!

### DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

## MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

### VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

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