## EN Seek® & Sable™ Seek® Caries Indicator & Root Canal Locator

### Description:

Seek® caries indicator contains D&C dyes in a glycol base that stain carious dentin red. Sable™ Seek® caries indicator contains FD&C dyes in an aqueous glycol base that stain carious dentin greenish black.

### Indications:

For practical purposes, Seek and Sable Seek will identify caries in difficult-to-see places; for example, under overhanging enamel of a Class I, II, or III preparation or along the DE junction of the preparation. They will prevent over-excavating deep caries. As deeper areas may be able to remineralize, some carious pulp exposures may be prevented. Seek is also used as a fast, easy way to locate the root canal orifice.

### Procedure:

- If using the IndiSpense® System, proceed as follows:
  - A pre-labeled unit dose syringe is attached to the end of the large, no-waste IndiSpense syringe by turning the luer lock of the unit dose syringe snugly onto the male thread of the IndiSpense, as shown in Fig. 1.
  - With the palm grasp, the plunger of the IndiSpense syringe is gently depressed while the unit dose syringe is stabilized at the plunger with the nondominant hand (Fig. 2). Slightly pull out plunger of unit dose syringe while expressing IndiSpense. Do not express contents of unit dose syringe back into IndiSpense. To prevent cross-contamination, a use syringe should not be reattached to the IndiSpense syringe.
  - Place a luer lock cap on unit dose syringe or attach a Mini Brush tip (use with an Ultradent® TipSoC® cover if not using immediately).
  - When ready to use, remove luer lock cap and twist the Mini Brush tip firmly onto syringe (or remove TipSoC cover if tip has been previously attached and capped). For optimum control, hold syringe with plunger in palm of hand rather than in conventional manner with thumb on plunger.
  - Point Seek or Sable Seek on surfaces to be checked. Leave for 10 seconds, then rinse and vacuum (Fig. 4).
  - When Seek is used to identify root canals in the early stages of an endodontic procedure, a small red dot will remain at the opening of the canal orifice.
  - For caries removal, use a slow-speed round bur or excavator (Fig. 5). Repeat as needed until caries is completely removed (Fig. 6).
  - Restore area with restorative of choice. We recommend PQ1® or PermaQuick® bonding agent(s) followed by Vit-I-essence® or Amelogen® composites.
- Caution: Chemical may set in tip after a few hours. To avoid clogging, remove tip and re-cap with Luer Lock Caps between each use. Always verify material flow prior to applying intraorally. If resistance is met, replace tip and re-check. This does NOT apply to dual-barrel syringes and mixing tips.
- Tips are disposable.
- Avoid splattering. Stain is removed from carious areas mechanically. If dye is diffusing to remove (intraorally) from areas other than caries, use a sodium hypochlorite solution such as ChlorCid® or use hydrogen peroxide.
- Use care when handling Seek, as it may stain clothing. If Seek contacts clothing, remove with detergent and warm water or use white vinegar\*.
- Clean and disinfect syringes between patients. We highly recommend using Ultradent® Syringe Covers sealed with the Ultradent® Impulse Sealer as a protective barrier to prevent cross-contamination.

### Precautions:

- Caution: Chemical may set in tip after a few hours. To avoid clogging, remove tip and re-cap with Luer Lock Caps between each use. Always verify material flow prior to applying intraorally. If resistance is met, replace tip and re-check. This does NOT apply to dual-barrel syringes and mixing tips.
- Tips are disposable.
- Avoid splattering. Stain is removed from carious areas mechanically. If dye is diffusing to remove (intraorally) from areas other than caries, use a sodium hypochlorite solution such as ChlorCid® or use hydrogen peroxide.
- Use care when handling Seek, as it may stain clothing. If Seek contacts clothing, remove with detergent and warm water or use white vinegar\*.
- Clean and disinfect syringes between patients. We highly recommend using Ultradent® Syringe Covers sealed with the Ultradent® Impulse Sealer as a protective barrier to prevent cross-contamination.

\*White vinegar has been proven effective for removing Seek/Sable Seek dye from garments, counter tops, covers, carpet, etc., regardless of color.

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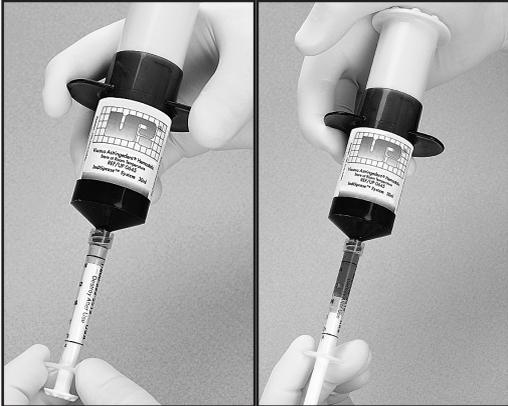
Figs. 1&2 A 1.2ml unit dose syringe is twisted firmly onto the IndiSpense syringe. While supporting plunger of unit dose syringe with the non-dominant hand, depress plunger of IndiSpense and fill unit dose syringe to desired level.

Fig. 3 Identify caries in difficult-to-see places.

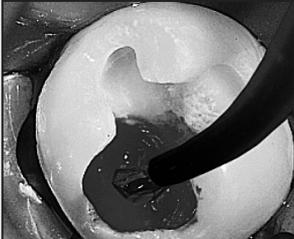
Fig. 4 Apply Seek to carious dentin with a Mini Brush tip. Rinse with air/water spray while vacuuming.

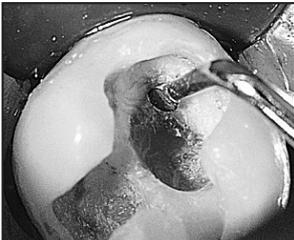
Fig. 5 Carious dentin is removed with a slow-speed round bur or excavator.

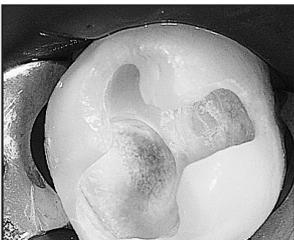
Fig. 6 Reapply indicator. Rinse and verify complete caries removal. A tiny amount of Ultra-Blend® plus™ is applied to deepest portion and light cured. Restorative material is placed.

General Precautions:
1. Caution: Chemical may set in tip after a few hours. To avoid clogging, remove tip and re-cap with Luer Lock Caps between each use. Always verify material flow prior to applying intraorally. If resistance is met, replace tip and re-check. This does NOT apply to dual-barrel syringes and mixing tips.
2. For Professional use only.
3. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
4. Keep products out of heat/sunlight.
5. Avoid skin exposure to resins.
6. Isolate strong chemicals to area of treatment.
7. Confirm that patient has no known allergies to treatment materials.
8. Test flow of materials from syringe and tip before using intraorally.
9. Never force syringe plungers.

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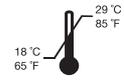
HAZARD RATING  
 4 = Severe  
 3 = Serious  
 2 = Moderate  
 1 = Slight  
 0 = Minimal

EN- Flammable  
 DE- Entzündbar  
 FR- Inflammable  
 NL- Brandbaar  
 IT- Inflammabile  
 ES- Inflamable  
 PT- Inflamável  
 SV- Brandfarlig  
 DA- Brandbar  
 FI- Syttyvä  
 EL- Εσφλεκτό  
 RU- Легковоспламеняющееся вещество  
 PL- Produkt łatwopalny  
 TR- Yanıcıdır  
 AR- قابل للاحتراق  
 KO- 인화성  
 ZH- 易燃  
 TH- ติดไฟได้

Ultradent Products GmbH  
 51143 Cologne  
 Germany

For product SDS please see our website: [www.ultradent.com](http://www.ultradent.com)  
 1-800-552-5512; 901-572-4200

EC REP 0044

U.S. Federal law restricts the sale of this device by or on the order of a dentist.  
 Keep out of reach of children.  
 For immediate re-use and/or complete descriptions of Ultradent products, please refer to Ultradent's catalog or call Toll Free 1-800-552-5512.  
 The first two numbers are the month, and the third number is the last number of the year.  
 Outside U.S. call (801) 572-4200 or visit [www.ultradent.com](http://www.ultradent.com).

Seek® & Sable™ Seek®  
 Caries Indicator & Root Canal Locator

Manufactured by:  
 Ultradent Products, Inc.  
 336 West 10200 South  
 Uauia, Utah 84906 USA  
 Made in USA

ULTRADENT  
 PRODUCTS, INC.  
 © Copyright 2019 Ultradent Products, Inc. All Rights Reserved.  
 11054-013 08/219