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---

### ENG - Choice of materials for production of Relax:

- Relax moulds (size 1, S1 and size 2, S2) are manufactured from polymethylmetacrylate (PMMA), which also forms the base for full dental prostheses.
- Relax Adhesive is a bonding agent of the ethylacetate type for joining silicone and PMMA material.
- Relax Fitting paste is a vinylpolydimethylsiloxane with a filler of the A-silicone family.
- Relax Cleaning spray is a fast, simple and effective spray with a pepper mint taste for daily cleaning of bite splints.

All materials have been tested in terms of being able to work in an oral environment. When used for the purpose intended and in accordance with the instructions, there are no known sideeffects linked to the material. For more information, see the respective product safety sheet.

### Indications for use of Relax:

- Patients with pain reported from the masticatory musculature, and verified palpation tenderness in the masticatory musculature (diagnosis myofascial pain).
- Patients 18 years of age and above

### Contra-indications for use of Relax:

- Open bite (teeth 13-23). Note that an open bite may have both a positive and negative overbite.
- Periodontally compromised dentition. Observe caution in cases of periodontally weak bites.

### Use

Relax have been tested for long-time use and is to be worn at night or half days.

When the splint is not being used it should be kept dry in its jar. Hygiene care of the splint is done using Relax Cleaning Spray and/or a soft toothbrush.

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#### ENG – Procedure

1. Try out the Relax mould (size S1 or S2) that suits the patient, i.e. ensure that the splint covers at least the canines both buccally and palatally when the central groove buccally is placed between the incisors in the upper jaw. Note that Relax is intended only for use in the upper jaw. (IMAGE 1)

2. Ensure in particular that the Relax mould does not lie palatally against the mucous membrane or tooth. If the mould does lie against it, grind it away with a rose bur.

3. Clean the Relax mould by degreasing it using Relax cleaning spray and then dry it out. Impurities can decrease the bonding between the Relax Fitting Paste and the prefabricated Relax mould.

4. Apply the Relax Adhesive evenly in a thin layer on all the clean and dry areas of the mould against which the Relax Fitting Paste is to be attached. Do not forget the edges. Apply twice within about 30 seconds so that all areas are covered. Relax Adhesive dries in about 1.5 minutes. Continue working only once the bonding agent has completely dried. (IMAGE 2)

5. Fill the whole of the Relax mould with Relax Fitting Paste. Be careful and apply Relax Fitting Paste so that no air bubbles remain under or in the material (for the best performance, squeeze out and discard a small amount of paste, before using a new tube). Relax Fitting Paste is adapted for application in combination with the Relax Fitting Paste "Pistol" or other pistol (DS50 1:1). It is a good idea to have Vaseline to hand when starting the process of filling the splint as Relax Fitting Paste can adhere to gloves. (IMAGE 3)

6. Place the filled Relax mould in the upper jaw, with the groove in the centre between the incisors in the upper jaw. Relax Fitting Paste. Press the splint into place. It may be an

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advantage to press the Relax up high using a pair of fingers and then place the splint as far into the mouth as the tooth arch/splint allows with a view to user comfort. Allow the patient to bite lightly together on the splint so the position is the most favourable with a view to the contacts with the splint. Allow the splint to remain in place for 90 seconds. During the fixed poly-merisation, a harmless increase in temperature occurs. Shape the Relax Fitting Paste on the palatal side of the splint against the palate using a gloved finger with vaseline. (IMAGE 4 & 5)

7. When the Relax Fitting Paste has stiffened after about 1.5 minutes, take away the Relax bite splint by grasping the buccal projections. (IMAGE 6)

8. In order to make use of the physical properties of Relax Fitting Paste in the best manner possible, you should allow a waiting time of about 10 minutes between hardening and continuing to work on it. Afterwards, cut away the excess using a scalpel, preferably a straight no.11. Be aware of unconnected streaks between the teeth which are in the inner material. If there are any such streaks, cut them away by using a scalpel. (IMAGE 7)

9. Try the splint and ensure the inner material does not lie against the gums buccally. Cut away any material that may be lying against it using a scalpel.

10. Grind in the splint from regio 13-23 to obtain a stable occlusion against 33-43, alternatively, multiple tooth contact with 33, 31, 41 and 43. When sliding to the side there should be contact with 33 and 43, alternatively with cuspids and incisors with as flat a slide as possible. The protrusion contact should be symmetrical about the centre line. *Note that Relax should not be ground so that it becomes less than 22 millimetres in a bucco-lingual direction.*

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## ENG – GRINDING

1. Try out the right size for Relax. It should cover the buccal surface from 13 to 23. If the splint is in too close contact with a tooth or the mucous membrane in the mouth, grind it with a rosen cutter to make it fit better.
2. After any grinding, check to make sure that there is "space" for individual adjustment using the recommended inner material Relax Fitting Paste.
3. After individual adjustment and removal of excess inner material (Relax Fitting Paste) with a scalpel, grinding of the splint can begin.
4. If some teeth do not make contact when biting in the retruded position (picture a) strive to achieve this (picture b).
5. Some bites may make it difficult for all of the teeth (33-43) to have contact with Relax. However, strive to at minimum make contact on both sides of the midline, preferably with 43 and 33 and 2 incisors.

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6. In case of lateral sliding, there should be contact with the corner tooth.

7. It is completely acceptable for contact with lateral gliding to be in the area of the corner tooth including incisors on the laterotrusion side. (pictures a, b)

8. It is completely acceptable to have contact with the corner tooth and the 1st premolar.

9. Occasionally there may be contact on both the laterotrusion side and the mediotrusion side. However, the mediotrusion side may not have more contact than the laterotrusion side.

10. If the Relax bite splint "comes loose" due to lateral sliding, change the angle on the splint so that the corner tooth glides on a more level surface.

11a, b, c. In protrusion, strive for even contact on both sides of the midline.

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