GC Fuji™ IX GP

RADIOPAQUE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT

For use by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Class I and II restorations in deciduous teeth.
- Non-load bearing Class I and Class II restorations in permanent teeth.
- Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- Class V and root surface restorations.
- Core build-up.

CONTRAINDICATIONS

- Direct pulp contact.
- Avoid use of this product in patients with known allergies to glass ionomer cement.

Directions for Use	
Powder / Liquid Ratio (g/g)	3.6 / 1.0
Mixing Time (sec.)	25-30*
Working Time (23°C/73°F) (min., sec.)	2'00*
Net Setting Time (37°C/99°F) (min., sec.)	2'20*

ISO 9917-1 Glass polyalkenoate cement / Restoration

1. POWDER AND LIQUID DISPENSING

- The standard powder to liquid ratio is 3.6 g/1.0 g. (1 level scoop of powder to 1 drop of liquid).
- For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- Hold the liquid bottle vertically and squeeze gently. Close bottles lightly immediately after use.

2. MIXING

- Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Mix the first portion with the liquid for 10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (Fig. 1).

3. RESTORATIVE TECHNIQUE

- Prepare the tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping, use calcium hydroxide.
- Apply CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge.
- Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening) (Fig. 2).
- Mix the required amount of GC Fuji IX GP Working time is 2 minutes from the start of mixing at 23°C (73°F). Higher temperatures will shorten working time.
- Transfer cement to the preparation using a syringe or other suitable instrument. Avoid air bubbles.
- Form the preliminary contour and cover with a matrix if required (Fig. 3).
- When set, immediately apply GC Fuji VARNISH or GC Fuji COAT LC (light cure) (Fig. 4).

4. FINISHING

- Final finishing under water spray using standard techniques can begin at 6 minutes after starting the mix (Fig. 5).
- Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.) (Fig. 6).
- Instruct the patient not to apply pressure for 1 hour.

STORAGE

Recommended for optimal performance, store at temperature of (4-25°C) (39.2-77.0°F).

SHADES

A2, A3, A3.5, B2, B3, C4 Shade numbers according to Vita® shade guide. ©Vita is a trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

- 1-1 package: 15 g powder, 8 g (6.4 mL) liquid, powder scoop, mixing pad (No. 22), plastic spatula
- Bottle of 15 g powder with scoop
- Bottle of 8 g (6.4 mL) liquid

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT mix powder or liquid with any other glass ionomer components.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gccorp.com> or for The Americas <http://www.gcamerica.com>

This can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

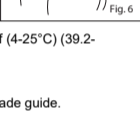
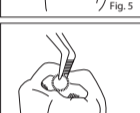
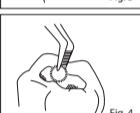
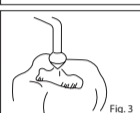
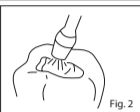
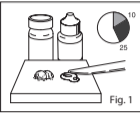
For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed/>) or contact us at Regulatory.gcc@gcc-dental

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gcc-dental. In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM
Coopers Court Newport Pagnell Buckinghamshire
MK16 8JS United Kingdom

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